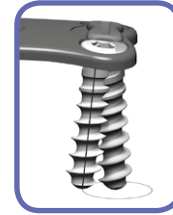
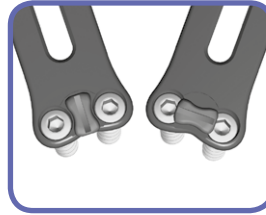
MORPHO

Plaque cervicale d'arthrodèse

Cervical arthrodesis plate

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Cervical arthrodesis plate system intended to provide a post-operative temporary stability to the cervical spine during the development of a solid spinal fusion.

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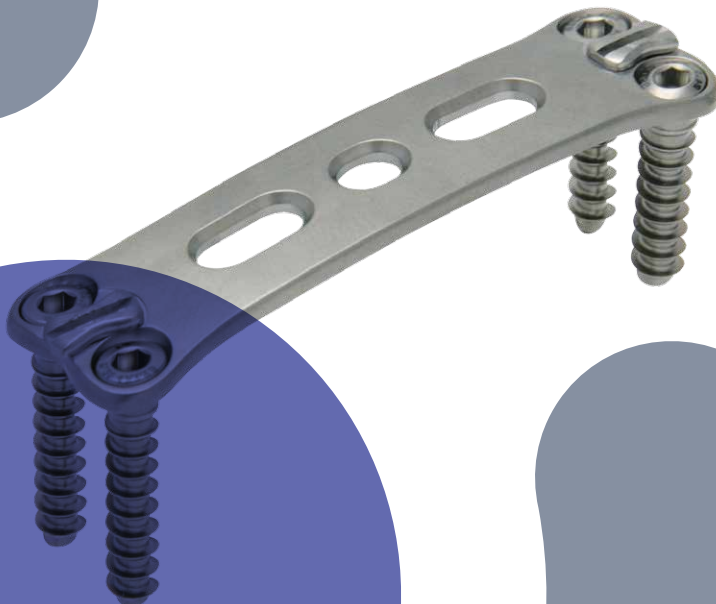
Features

- Anatomic: 8°
- Easy and safe: integrated locking button (90° rotation)
- Practical: multiaxiality screw Ø 4
- Complete range: Ø 4 & 4,5
27 Sizes - from 1 to 4 level configurations
- Low profile: 1.6 mm

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



Treatment of degenerative diseases of the cervical spine (C3-T1) including:

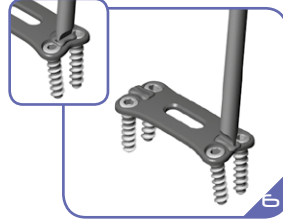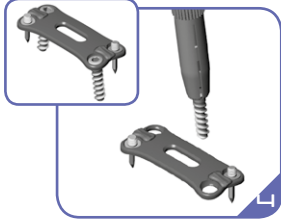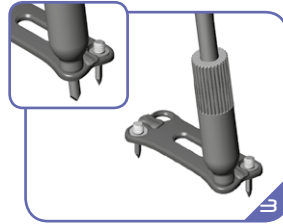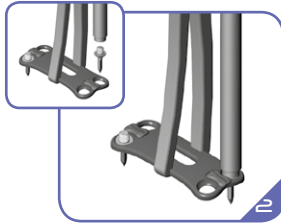- Degenerative pathologies
- Spinal deformity (defined as kyphosis, lordosis, or scoliosis)
- Traumatology (including fractures)
- Tumoral pathologies

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- 1 Define the size of the plate
- 2 Impact two pins in diagonal position
- 3 Set the sharp point in its site and push
- 4 Place two screws
- 5 Remove the pins and replace with screws
- 6 Turn 90° the locking button

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The screws of diameter 4 mm are variable angle screws.
The screws of diameter 4.5 mm are fixed-angle screws and are used if a rigid construct is desired or in situations where bone purchase has been lost with a 4 mm screw.

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MORPHO is a medical device CE marked of class IIb (implants), class I (instruments) manufactured by hpi and for which the conformity evaluation was undertaken by the notify body SGS n°1639.
Devices availability may vary according to countries and markets.
This document is intended for healthcare professionals only. Before any surgical procedure, please read the instructions of use carefully.